

# INSTALLATION GUIDE

## ASPIRE CONFIMOTION BED COLLECTION

This guide is intended to explain the correct set up procedures for the **Aspire ComfiMotion Bed Collection** upon installation. For further installation and operating instructions, refer to the product **User Manual**.

<b>1. ADJUSTABLE BEDS</b>	LUXE (KS) BEB046710	CARE (LS) BEB046720	CARE (KS) BEB046730
<b>2. HEADBOARDS BEDS</b>	ALLURE (LS) BEA830120	CHESTERFIELD (LS) BEA830100	
	ALLURE (KS) BEA830130	CHESTERFIELD (KS) BEA830110	

### ASSEMBLY OF CONFIMOTION BED (CARE AND LUXE)


- 1.1** Remove bed from carton and position on flat surface or a Bed Moving Trolley for easy setup. Ensure the bed is placed **TOP DOWN**, so all components (hand control, retainers, castors & power cord) can be removed easily from the underside of bed.



When opening the carton, ensure it is done in a safe and careful manner. **DO NOT USE** a box cutter or sharp implement as this may damage the product.

- 1.2** Using a box cutter, unhook all components from the base of the bed (retainers, power cord, hand control and castors).

**Retainers**

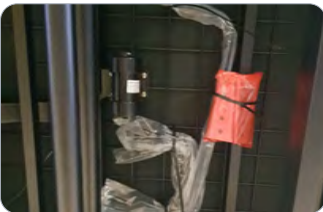

**Hand Control**



**Castors**



**Power Cord**



- 1.3** Install castors onto all 4 x corners of bed, securing in place with a 12mm spanner. **NOTE:** 75mm Castors shown. When fitting larger (MWT451080 - Optional Accessory) castors, the process is the same.



- 1.4** Connect hand control into control box port (**CARE bed only**).



**NOTE:** LUXE Bed has a wireless Hand Control, no need to connect to Control Box.

- 1.5** Cut safety securing straps that hold the bed in place when in carton.



- 1.6** Turn bed FACE UP, secure 4 x mattress retainers (side) and 1 x mattress retainer (foot) in place by pushing into allocated slots.

**Side Mattress Retainer**



**Foot End Retainer**




- 1.7** Place mattress on bed, it is now ready for use.



## ASSEMBLY OF CONFIMOTION HEADBOARDS (ALLURE & CHESTERFIELD)

- 2.1** Remove headboard, brackets and mounting screws from carton.

- 2.2** Turn headboard upside down to show black upholstered section and align bracket holes with white markings. Secure brackets in place using and impact driver/drill with mounting screws provided.

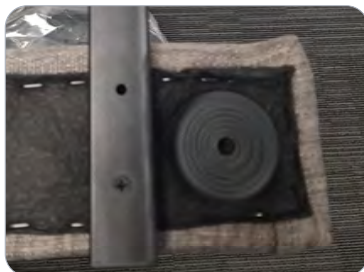

- 2.3** The headboard will now stand on its own, place securely behind bed ready for use.



## ASSEMBLY OF HEADBOARD TO BED - LUXE ONLY

For further installation and operating instructions, refer to the **User Manual**.

### Tools Required:

- Impact driver or drill (5mm attachment)
- Phillips head screw driver (5mm)

**NOTE:** It is suggested that an impact driver or drill is used to speed up this process. Using a Phillips head screwdriver will work but will take more time. If the impact Driver/Drill has a clutch setting, it should be set to low torque to minimise any risk of bolt misalignment or mount point damage.

**NOTE:** Headboards can be mounted to the **LUXE Bed only**.

**CAUTION:** When bed is in raised position, Headboard will raise with bed and poses risk to damaging of fixings or attachments on walls.

- 1 Raise bed height, backrest and knee break to maximum height for ease of access.

### LUXE Handset

**Hi** Bed Height

 Backrest

 Knee Break




- 2 Locate mounting holes for screw insertion. To locate holes:

- a. Locate red dots on bed frame (looking top down at head section of bed).
- b. Using an impact driver, rotate screw through the upholstery inwards towards frame.
- c. Through rotation the screw will pierce through the fabric and thread into wood frame (self-tap). Repeat on both sides, ensuring 4 x screws (2 per side) are fastened in place.
- d. **NOTE:** Position of bracket should be in line with red dot and spaced approx. 25mm between the top and bottom of bed frame.



**3** When mounting brackets to bed end:

- a. Line bracket up with foot stopper (on base of headboard).
  - i. Ensure bracket is spaced approx. 40mm from edge of headboard.
  - ii. Ensure bracket is spaced approx. 60mm from base of headboard.
- b. Through rotation the screw will pierce through the fabric and thread into wood frame (self-tap). Repeat on both sides, ensuring 4 x screws (2 per side) are fastened in place.
- c. Using an impact driver, rotate screw through the upholstery inwards towards frame.
- d. **NOTE:** Position of bracket should be as per measurements otherwise it will not fit securely to the bed frame.

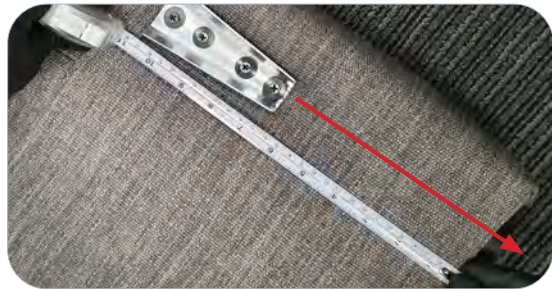

**4** Align bracket fixings (headboard bracket will slide into bed frame bracket), Slide headboard down until both brackets click into place, the headboard is now in place.

**NOTE:** Headboard foot supports will rest on the ground when in lowest height.



# ASPIRE COMFIMOTION BED - ACCESSORIES

For further installation and operating instructions, refer to the **User Manual**.

## Tools Required:

- Allen Key (5mm)
- Impact driver or drill (5mm attachment)


**NOTE:** It is suggested that an impact driver or drill is used to speed up this process. Using a Phillips head screwdriver will work but will take more time. If the impact Driver/Drill has a clutch setting, it should be set to low torque to minimise any risk of bolt misalignment or mount point damage.

**1** Raise bed height, backrest and knee break to maximum height for ease of access.

### LUXE Handset





### Care Handset

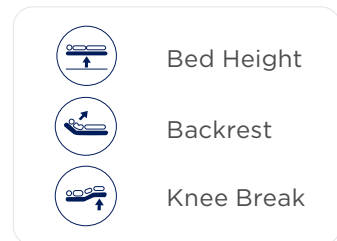

**2** Remove head end mattress retainer (bed rail only), the rail secures the mattress in place.




**3** ComfiMotion beds have pre-installed mounting points for accessories.

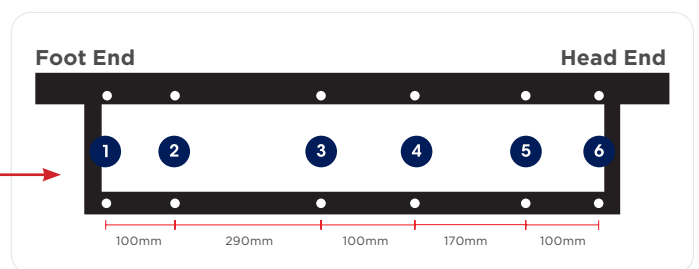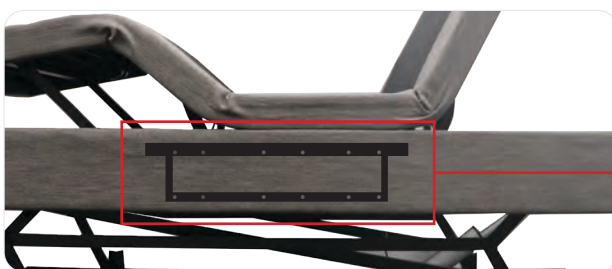


**Note:** Frame not visible through upholstery, for instructional purposes only.

## To install Bed Rail (BEA830210):


### Position 1 = Use Holes 1 & 5

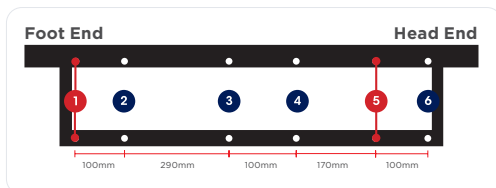

### Position 2 = Use Holes 2 & 6

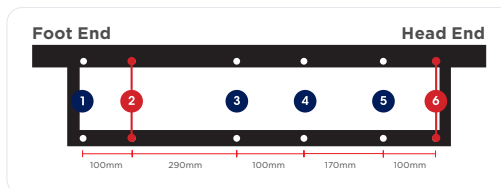


Bed Rail

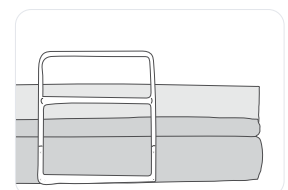


**To install Crook Handle Bed Stick (BEA830200):**


**Position 1 = Use Holes 4 & 6**

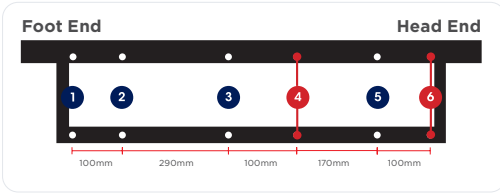

**Position 2 = Use Holes 3 & 5**

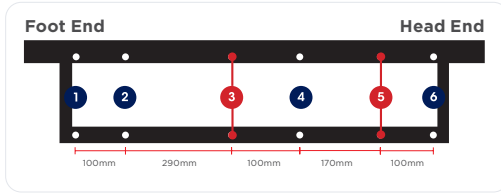

The handle can be mounted facing in either direction.

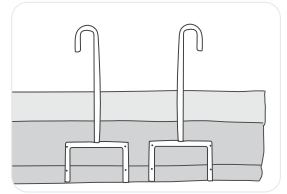


- 4** Locate mounting holes for bolt insertion. To locate holes:
- Press on upholstery in line with frame hole and the threaded hole insert can be felt with fingers.
  - Using an impact driver or allen key, rotate bolt through the upholstery inwards towards frame.
  - Through rotation the bolt will pierce through the fabric and thread into pre-installed frame thread.

**Using an impact driver/drill**



**Using an allen key**

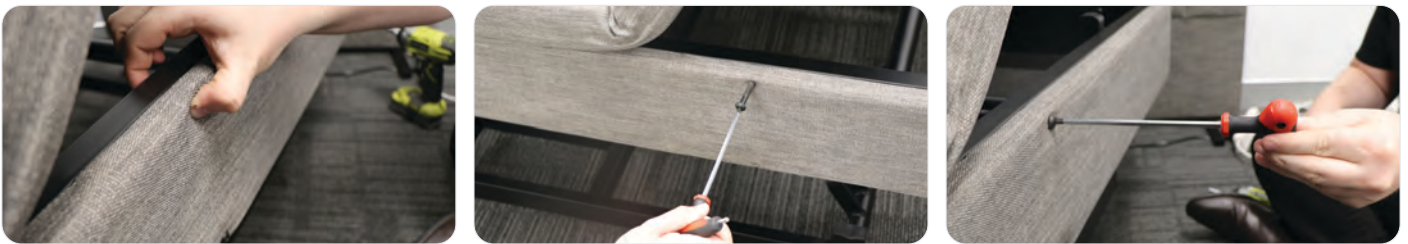

- 5** Repeat this on all 4 x mounting points (x2 top & x2 bottom) to create guide positions. Ensure bolts are removed so as to start next step of fastening the accessory.
- 6** Align bed pole or rail with side of bed, using 4 x guide holes, screw in bolt using an impact driver/drill or allen key until fully fastened.



**Note:** As bolts pierce upholstery, holes need to be covered up. If bed rail or pole needs to be repositioned, use (M8 x 60mm) and fasten in place, this will remove the exposed hole and tidy up the side of the bed. **Bolt kits available:** Item Code: GSP640110.

