



aspireswift PATIENT MOVER

The Aspire Swift Patient Mover is a compact standing aid that facilitates standing and transfers in the home, community and facility environments. Offering a more active transfer solution, users are encouraged to stand up independently, improving circulation, breathing and muscle strength. The Aspire Swift Patient Mover is an ideal alternative to a patient lifter for users with appropriate sitting balance, who require transfers over greater distances.

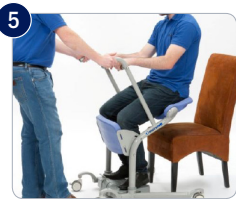

1	Four user contact points (foot platform, knee pad, seat pads and grip handle) promote stability, support and comfort for the user - resulting in greater confidence in the transfer	6	Lockable 75mm castors - the platform remains stationary while a user is transferring, while minimising the leg height for sufficient clearance beneath beds and furniture
2	Rigid foot platform - provides a stable surface for the user to stand on and support their own weight	7	Width adjustable legs activated via 2 foot pedals, provide the ability to manoeuvre closer to furniture, toilets or beds for safer access
3	Height adjustable soft knee pad - provides comfortable counter balance stability during transfers	8	Clip-sling attachment points - allows a standing sling (optional accessory) with clips to be used for additional support where required
4	Soft seat pads - swing out of the way to allow user access and swing back into place to provide a comfortable seating surface	9	Easy to clean surfaces - for effective and hassle-free infection control
5	Soft touch grab handle - assists the user to move independently from sitting to standing		

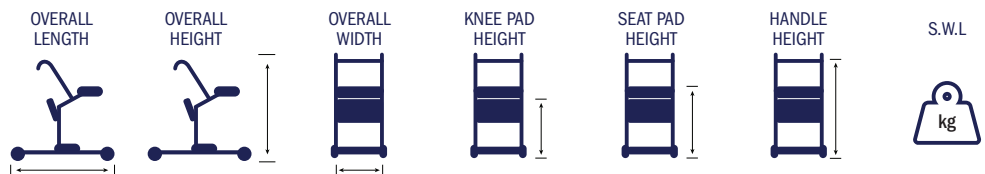

LSS390590 Swift Patient Mover	OVERALL LENGTH	OVERALL HEIGHT	OVERALL WIDTH	KNEE PAD HEIGHT	SEAT PAD HEIGHT	HANDLE HEIGHT	S.W.L
	820mm	1060mm	490 - 760mm	250 - 490mm	620 - 670mm	900mm	185kg



HOW TO USE

- ✓ Swift Patient Mover is registered in the ARTG, AUST R ARTG 265455
- ✓ Tested to International Standards



Open legs if required and lift seat pads. Ask the user to place feet on foot plate and apply brakes.




Ask the user to hold the grab handle and to stand up.



Swing the seat pads back into place and ask the user to sit down. Unlock brakes and carry out the transfer.

SLING COMPATIBILITY

Aspire Swift Patient Movers feature clip-sling attachment points to allow standing slings (optional accessory) to be used for additional support and comfort. Below is a guide to the Aspire Standing Clip Sling range and sizing guidelines.

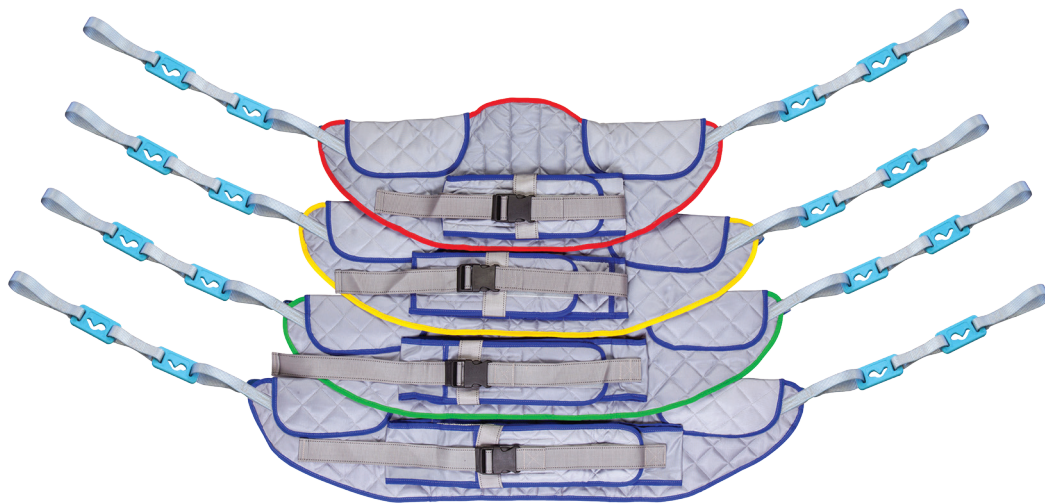

DELUXE STANDING SLINGS

Code	Description	Girth	Edging	S.W.L
LSS390920	Deluxe Padded Standing Sling - Polyester - Clip - Small	65–95cm	●	200kg
LSS390925	Deluxe Padded Standing Sling - Polyester - Clip - Medium	77–110cm	●	200kg
LSS390930	Deluxe Padded Standing Sling - Polyester - Clip - Large	98–135cm	●	300kg
LSS390935	Deluxe Padded Standing Sling - Polyester - Clip - Extra Large	118–157cm	●	300kg

For men and children, girth is the chest circumference. For women, girth is the hip circumference.